MoVeS

Cuff-Ring

TUBE

en – Medical Device – Instructions for use

Intended purpose: To tone, shape, strengthen or rehabilitate muscle groups through resistance exercise, for use in physical therapy or fitness training. **Intended user:** Lay person or healthcare professional. **Indications:** Medical conditions resulting in limitation in movements, postural impairments and muscle performance impairments. **Contra-indications:** Use should be discontinued in case of unusual swelling, discomfort, dizziness, severe muscle or joint pain or pain in the chest.

- Precautions:**
- Consult a healthcare professional before starting an exercise programme;
 - Always examine MoVeS Ring Tube before use for cuts, small tears or punctures. Do not use if damaged;
 - Store away from sharp objects. Remove rings and be aware of sharp fingernails;
 - Store away from sunlight.

Warnings:

- This device contains natural rubber latex which may cause allergic reactions;
- Do not stretch beyond 150cm.

Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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